

# PERFORMING A RESCUE

1



Is patient responsive?

2



Call emergency services!

3



Get AED.

4



Place AED on ground.  
Open lid.

5



Remove clothing.  
Expose chest.

6



Attach pads.

7



Do not touch the patient.

8



If instructed,  
press red flashing button.

9



If present, place CPR device.

10



Follow AED prompts.  
Give compressions.

11




If instructed, give breaths.

## Semi-automatic AED



Cardiac Science and the Shielded Heart logo are trademarks of Cardiac Science Corporation.  
Copyright © 2019 All Rights Reserved.  
500 Burdick Parkway, Deerfield, Wisconsin 53531 USA  
[www.cardiacscience.com](http://www.cardiacscience.com)

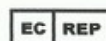

MDSS GmbH  
Schiffgraben 41  
D-30175 Hannover  
Germany



70-01972-01 A

